

1) COMPONENTS / COMPONENTES / KOMPONENTEN /
COMPOSANTS / COMPONENTI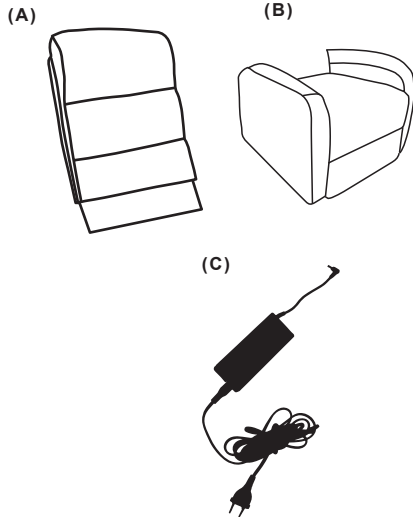

2) ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO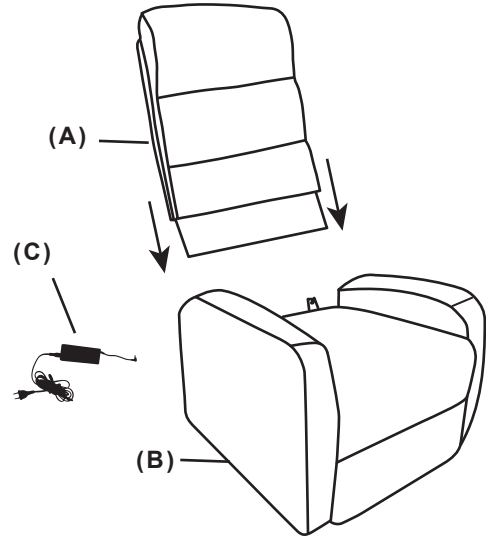

EN

CODE

PEPE - POWER LIFT CHAIR (BEIGE)	P40028	PEPE - POWER LIFT CHAIR (BLUE)	P40029
---------------------------------	--------	--------------------------------	--------

The **PEPE Lift Armchair** is an armchair designed to help people to stand up easily. It includes a control that allows both recline the seat and incorporate it. Maximum user weight: 287 lb.

ASSEMBLY

1. Insert the backrest (A) through the pins on the back of the seat (B) and attach the Velcro flap on the back of the seat.
2. Connect the power cord to the light to be able to use the lift chair.

MATERIAL

- Seat: foam (density: 30 kg/m³).
- Backrest: microfiber padding (density: 30 kg/m³).
- Fabric: linen microfabric, 100% polyester.

WARNINGS!

- The armchair must be placed on flat ground.
- Do not use the armchair in a damp or dusty environment, as this may cause an electric shock or problems with the machine. The chair should be in a dry room.
- Do not use the chair at a temperature above 40°C.
- Do not expose your chair to heaters, stoves or direct sunlight.
- Do not use the chair outdoors. Indoor use only. Avoid exposure to rain, snow, ice, salt or standing water. Keep and store in a clean, dry place.
- When the chair is not in use, disconnect the power cord from the mains to prevent overheating.
- Do not rest or place heavy objects on the armrest, footrest or backrest of the chair.
- Do not jump or play on the chair.
- Do not sit or stand on the footrest.
- Do not sleep in the recliner during operation.
- Children must be supervised by an adult when using the recliner.
- Keep pets away when the recliner is in motion.
- Check that the voltage corresponds to the stated specifications.
- The electrical connection must be carried out in accordance with local regulations.
- Do not connect or disconnect the plug from the power outlet with wet hands. It may cause electric shock or malfunction of the machine.
- When disconnecting the plug, pull it out by holding it by the plug and not by the cord.
- Do not place the AC/DC adapter cord under the chair, as this may damage it and cause a fire or electric shock.
- Do not touch the surface of the plug immediately after use, as it may be very hot.
- Do not operate the remote control with wet hands or with sharp or heavy objects.
- Do not insert fingers or hands between the seat and footrest when the recliner is reclined or is returning to the home position.
- Do not remove the sticker provided on the product. If necessary, note the batch of the product so that the manufacturer can be notified in the event of an accident.

CLEANING AND DISINFECTION

The product can be cleaned with a damp cloth. Do not use bleach or any other corrosive product.

WARRANTY INFORMATION

The company warrants this device to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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