EN	CODE
PEPE - WALKER WITHOUT WHEELS	P10012

The **PEPE WALKER WITHOUT WHEELS** is a lightweight aluminum device, foldable and adjustable in height, with great stability and resistance. Its front bar is made of steel. Recommended for people with reduced mobility.

#### FOLDING SYSTEM

Remove all packaging and examine all parts for signs of shipping damage. Do not use the walker if it is damaged or missing parts.

#### To deploy:

- Open the legs back until you hear a click on the central adjustment (1).
- Block the rattle with the adjustment clip located on the front tube (2) by removing it and placing it in the side hole (3). Do it on both legs.
- Cover the locking clip with the trim (4)
- Adjust the height of the walker with the available regulations and make sure that it is the same in all the legs (5).

#### To fold:

- Remove the two side adjusting clips and reattach them to the head tube.
- Remove the bezel and squeeze the clip while pushing the legs inward.

#### ADJUST THE HEIGHT

1. Stand tall next to the walker with your shoulders relaxed and your arms slightly placed at 15°.
2. Adjust the height so that the grips are at wrist height.
3. Set the correct height using the 8 adjustments.
4. Check that both legs are at the same height.

#### WARNINGS

- This is a walker that enhances the restricted ability to move indoors and outdoors.
- Make sure the walker is fully open and in the correct position before use. Make sure it is properly adjusted to the correct height.
- Check the stability of the walker before using it.
- This walker should not be used as a transport device for people or objects.
- Use only on level surfaces, never on a slope or uneven ground.
- Do not stop or turn on a ramp or incline.
- Do not exceed the maximum user weight: 220 lb.
- Do not expose the product to temperatures above 104 ° F for a long time; otherwise the aluminum tubes will become hot and could cause burns.
- You should only begin to move with your walker when you are comfortable standing and when you have both hands on the handles.
- Observe the care and maintenance instructions. The manufacturer assumes no responsibility for damage resulting from improper care or maintenance of the walker.
- Inspect the walker before each use to ensure there is no damage or excessive wear.
- Check that all end caps and folding parts move properly and that there is no accumulation of debris around the shaft.
- Keep your walker clean by using a damp cloth. Do not use abrasive cleaning materials.
- This walker is free from defects. The warranty will remain in effect for 2 years from the consumer's original purchase date. The warranty does not extend to parts subject to natural wear and tear that require periodic replacement, such as brakes, wheels, handles and bags. This warranty does not cover:
  - Spare parts supplied by anyone other than the authorized dealer.
  - Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- In case of return, pack the walker properly, preferably in its original packaging, otherwise the company will not be responsible for possible damages caused during shipping.
- This does not affect your legal rights.
- Keep these instructions for future reference.
- According to the manufacturer's recommendations, this product is suitable for recycling.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

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