EN

CODE	
PEPE-WALKER	P10003

The PEPE Walker is a sturdy and lightweight walker ideal for indoor use. It has padded handles that allow a more comfortable support and favour unloading avoiding discomfort in hands and wrists. It is designed to be used by elderly people or people with reduced mobility who need a walking aid.

FOLDING SYSTEM

Remove all packaging and examine all parts for signs of shipping damage. Do not use the rollator if it is damaged or any parts are missing.

When unfolding, pull the rear tubes backwards and push the attachment plates on both sides downwards until these locks are in position. Then, place the cushion on the seat and adjust the handles to the correct height.

When folding, remove the cushion from the seat and pull the attachment plates upwards. Keep fingers away from the folding mechanism. Then push the rear tubes forwards against the front tubes.

HEIGHT ADJUSTMENT

1. Stand straight behind the walker with your shoulder relaxed and arms slightly bent at 15°.
2. Set the handle height to be level with your wrists.
3. Set the correct height placing the handle height adjustment.
4. Follow the same steps to adjust the height of the other handle and check that both hands are adjusted to the same height.

BRAKE SYSTEM

The walker has a pressure brake system that is activated by the user's own weight and helps control the movement of the walker. To brake, grasp the handles and, with the weight of the body, push down until the walker stops.

WARNINGS

- This is a walking device for compensation and improvement respectively of restricted ability of walking.
- Ensure the Walker is fully open and secured in the correct position before use.
- Ensure the handle is set to the correct height and fully secured.
- Test the Walker for stability before use.
- Test the brake mechanism before use.
- The Walker is a walking aid only and should not be used as a transportation device. Use only on even and solid surfaces, never on an incline or uneven ground.
- Do not stop, turn or reverse on a ramp or incline. Do not make sharp turns.
- Do not exceed the maximum user weight: 220 lbs.
- Do not expose the product to temperatures above 104 °F for a long time, otherwise the aluminium tubes cover heat, which could lead to burns.
- Ensure there are no objects blocking the wheels of the rollator.
- Only proceed moving with your rollator when you are standing comfortably and with both hands on the handles.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.
- The Walker should be inspected before each use to ensure it is free from damage and excessive wear. If you do find wear or damage to your Walker, DO NOT USE.
- Check that both wheels, tips and folding parts move freely and that there is no build-up of debris around the axle.
- Keep the Walker clean by wiping with a damp cloth on regular basis. Do not use abrasive cleaning materials.
- The Walker is free from defects in material and workmanship. The warranty shall remain in effect for 2 years from the date of original consumer purchase. The warranty does not extend to parts that are exposed to natural wear and tear that will require periodic replacement such as brakes, wheels, handgrips and bags. This warranty does not cover the following:
 - Replacement parts supplied by anyone other than the stockist.
 - Any failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use.
- Should it be necessary to return the Walker, it is essential that it is adequately packed to protect it in transit, preferably in its original packaging, otherwise the company cannot accept any responsibility for transit damage to the unit.
- This does not affect your statutory rights.
- Please retain these instructions for future reference.
- The product is suitable for reprocessing in line with manufacturer's recommendations.
- Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.

The product can be **cleaned and disinfected** with commercially available products, as well as with a damp cloth. Always keep the product clean and wash it regularly.

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