CODE	
PEPE - WALKING STICK	P10004
PEPE - WALKING STICK BRONZE	P10005
PEPE - BASTÓN FLORES	P10008

The **PEPE Walking Sticks** are designed for people who need extra support for walking. They have a simple folding system in four sections for easy storage and transport of the cane.


#### HEIGHT ADJUSTMENT

Do not use this equipment without first reading and understanding these instructions. If needed, contact health care professional, dealer or technical personnel before attempting to use, otherwise injury may occur.

Adjust the height following the next instructions:

- Stand on a flat surface and let your arm hang down naturally by your side. Set the cane height so the top part of the handle touches the inside part of the wrist.
- Release the safety locking thread (1) by turning it clockwise once. Press the clip (2) and slide the tube up or down between the 5 settings to the appropriate height.
- To set the height make sure that the pin is correctly placed in the hole and place the safety locking band again.

When using the cane, **grip the handle placing its long part backwards.**

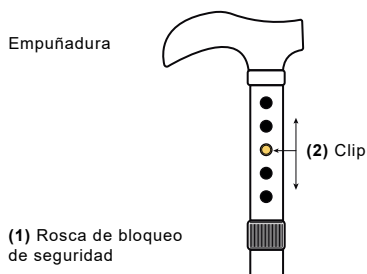#### GENERAL WARNINGS

Follow the instructions below to ensure safe use of the walking stick:

- Walking sticks must be regularly checked for any damage or wear.
- Hold the whole stick when unfolding it, so it does not spring out and cause any damage or surprise to the user or others around. Keep fingers clear when folding it, so it do not get caught in any way.
- Take care when using on slippery surfaces, a walking stick cannot stop you from slipping.
- Do not apply excess weight to the stick and never exceed the indicated maximum weight.
- Walking sticks are solely for use as an aid to walk and they are not designed to bear the total body weight.
- Before using the cane, check that all parts are connected and securely fixed. In case of signs of breakage or damage, do not use the cane.
- Check that the height adjustment clip is correctly positioned and securely locked in one of the holes.
- Check that the safety lock thread is tight when using the cane.
- Do not store the cane in temperatures below freezing.
- Use the cane only for the purpose for which it was designed. The company is not responsible in any way for any misuse of the cane, or for injuries or damages caused as a result of it.
- Any serious incident related to the product should be reported to the manufacturer and the competent authority of the Member State in which they are user and patient set.
- Keep the Rollator clean by wiping with a damp cloth on regular basis. Do not use abrasive cleaning materials.

#### TECHNICAL SPECIFICATIONS

MATERIAL	HEIGHT	WEIGHT
ALUMINIUM	32.6" - 36.2"	0.76 lb ± 0.52 Oz
FOLDING DIMENSION	MAX USER WEIGHT	Ø RUBBER END TUBE
11.8"	220 lb	5/8"

REFERENCIA	
PEPE - BASTÓN	P10004
PEPE - BASTÓN BRONCE	P10005
PEPE - BASTÓN FLORES	P10008

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