

1) COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI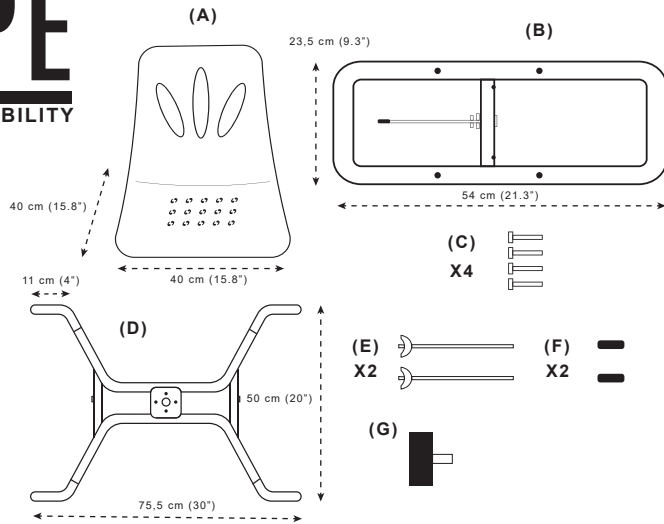

2) ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO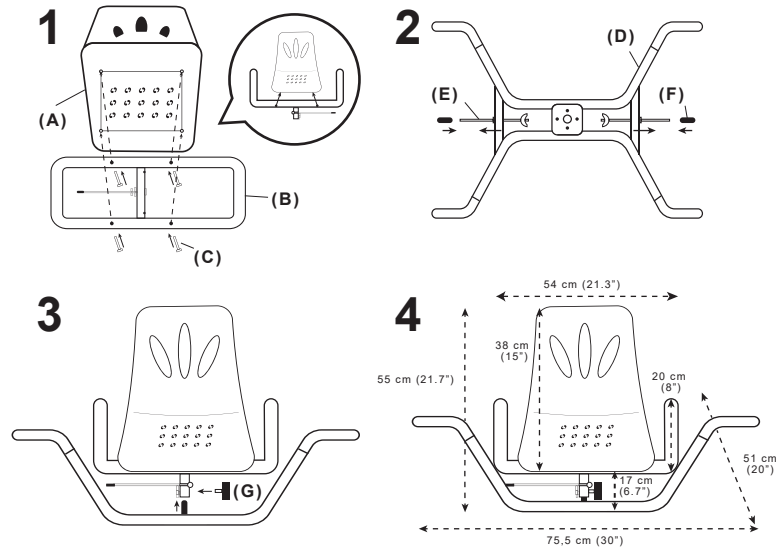

EN

CODE	
PEPE - SWIVEL BATH CHAIR	P30013

SWIVEL BATH CHAIR

Manual provided for the user of the product. BEFORE using this product, read this manual and keep it.

INDICATIONS OF USE

The PEPE SWIVEL BATH CHAIR is perfect for the hygiene of people with reduced mobility inside the bathtub. It is characterized by the ability to rotate its seat, which facilitates and ensures patient transfers.

FEATURES

- Steel structure painted with white epoxy and polyethylene seat.
- Its capacity is 220 lb and its weight is 10 lb.
- Surface with drainage holes.
- Safe and stable thanks to its non-slip feet.
- 360° rotation

ASSEMBLY

1. Turn over the seat (A) and place the frame with handles (B) by screwing it with the screws (C) in the holes for it as in Figure 1.
2. Put the frame with supports and swivel lever (D) so that it can be inserted into the safety fixing (E) with the gasket (F) on both sides (Figure 2). This wrench will allow the swivel chair to be securely adjusted inside the bathtub.
3. Finally (Figure 3), insert the upper part of the frame with supports and swivel lever (D) so that it fit with the frame with handles (B) and the seat (A) assembled in step 1. A Once fitted, adjust with the thread (G) prepared for it so that it looks like Figure 4.

WARNINGS!

- Make sure that the legs of the swivel chair are positioned securely against the inside wall of the bathtub and tighten the side safety fixings (E).
- Make sure the screws are tight on the chair.
- Do not stand up in the chair.
- Do not move or move the product while it is loaded or while in use.
- Do not stretch or move the removable parts to move the seat.
- Check the product each time before use.
- The user must be able to maintain balance in case of use without an assistant.
- Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.

CLEANING AND DISINFECTION

The product can be cleaned and disinfected with products available on the market. We recommend cleaning the product before use by another patient, after the end of the period of use and whenever it has been in contact with body fluids.

1. Use water and a mild non-abrasive detergent and wipe dry.
2. Disinfect by cleaning all accessible surfaces with disinfectant.

WARRANTY INFORMATION

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

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