

# Gel Air Ankle Brace

SKU: AKB173

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Fitting Instructions

**Step 1.** Open ankle stirrup and adjust heel strap.

**Step 2.** Place foot into brace and center the side stirrups on lower leg. Wrap lower strap snugly then wrap top strap, adjusting for a secure fit.

**Step 3.** For cold therapy, remove therapeutic bladders from stirrup and place in the freezer. Do NOT place entire ankle support in the freezer.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

