

Unisex Deluxe Post Op Shoe

SKU: POS294

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Fitting Instructions

Step 1. While seated, unfasten all straps. Place heel at the back of the shoe and secure the ankle strap.

Step 2. Fasten tow straps, ensuring toes are not over the end of the shoe. Adjust as needed to accommodate bandages or swelling. Do not over-tighten.

Step 3. Trim excess ankle strap as needed.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

