

Hot/Cold Pack Insert for Back Braces

SKU: ACS236

Prefer watching videos?
We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Remove the cold pack from the outer sleeve and place in the freezer for at least two hours.

Step 2. Insert the cold pack into the outer sleeve.

Step 3. Use the Velcro on the outer sleeve to attach it to the inside of your brace. When done using, replace the cold pack in the freezer for future use.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Spray with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.   

