

Ankle & Foot Reusable Ice Pack Cold Wrap

SKU: ACS272

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Prior to use, place the removable cold packs in the freezer for two hours.

Step 2. Unfasten Velcro, open the wrap, place the cold pack in the proper position, and place the foot into the wrap.

Step 3. Fasten the foot strap to comfortable compression.

Step 4. Fasten the ankle strap to secure it in place.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

