

Semi-Rigid Active Ankle Brace

SKU: AKB134

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Fitting Instructions

Step 1. Unfasten all straps and place the brace into the shoe.

Step 2. Spread the uprights and place foot into brace in the shoe.

Step 3. Fasten the velcro straps around the ankle until secure.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

