

Air Stirrup Ankle Brace

SKU: AKB172

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Open the ankle stirrup and adjust the heel strap. Step into the brace making sure the side stirrups are centered on the lower leg and ankle.

Step 2. Wrap the lower strap snugly around the brace. Wrap the top strap and readjust the lower strap to achieve a secure fit.



Step 3. Step into the shoe and tighten the laces.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Spray with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  



BraceDirect.com • 6860 Big Ridge Rd • Hixson, TN 37343 • (800) 257-5870