

Semi-Rigid Active Ankle Brace

SKU: AKB260

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Unfasten the two front velcro straps.

Step 2. Place brace footplate into the shoe.

Step 3. Place foot into shoe while spreading the side panels.
Fasten Velcro straps securely.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

