

# Pediatric Children's Arm Sling

SKU: ARS265

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Gently pull the sling over your arm and elbow. Your elbow should be snugly in the back of the sling. Your hand should be at the very end of the sling and should not cut into your wrist or hand.

**Step 2.** Reach around your neck and grab the strap behind your elbow. Pull the strap around the back of your neck and push it through the D-ring near your hand. Tighten the straps to keep your hand and forearm elevated above the level of your elbow.

**Step 3.** Secure the strap with Velcro and place your thumb into the thumb loop.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

