

Deluxe Shoulder Immobilizer

SKU: ARS268

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code




Instructions For Use

Step 1. Place injured arm in the sling and place thumb in the thumb loop.

Step 2. Place the strap around the neck, loop through the D ring and secure. Adjust for proper height and comfort.

Step 3. Place waist strap around the waist, loop strap through the buckle, and attach Velcro closure.

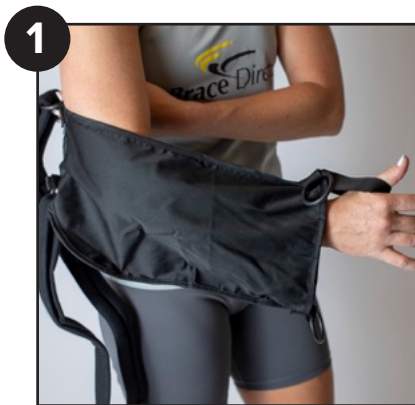

Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach. 