

Standard Shoulder Immobilizer

SKU: ARS269

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Place arm into sling envelope with the elbow snugly in the corner.

Step 2. Pull the shoulder strap over and behind the affected shoulder, then through the D ring. Secure the strap back onto itself with the Velcro. *Note: The elbow should be resting at a 90° angle.*

Step 3. Pull the waist strap through the side ring and secure it back onto itself with the velcro.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

