

SI Joint Belt

SKU: BKB-CL-SI-100

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Open the Belt

Unfasten the Velcro and open the belt. Center the seam in the middle of the back.

Step 2. Fit the Belt


Fasten the closure strap and ensure the belt fits snugly.

Step 3. Compress the Straps

Detach the side compression straps and pull both way from the body to achieve compression.

Step 4. Adjust Belt

Fasten the compression straps to the Velcro on the inside of the belt and adjust until the desired level of compression is reached.

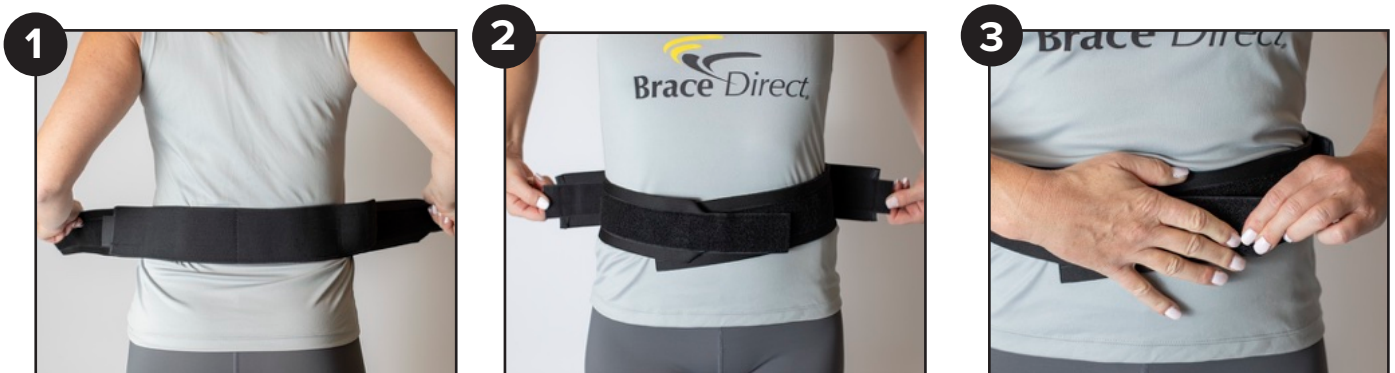

Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach. 