

# Low Back Support Belt

SKU: BKB123

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Open your back brace completely. Locate the Velcro closure on the waist panel.

**Step 2.** Attach the end of the extension belt to the velcro closure.

**Step 3.** Wrap around your waist and secure the waist panels with Velcro.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Spray with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.   

