

# Postural Extension Back Straightener Brace

SKU: BKB175

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Locate the left and right pulley handles and fasten them near the pulley unit to avoid tangling while fitting.

**Step 2.** While standing, hold the side panels apart and center the back panel along the spine just above the tailbone.

**Step 3.** Slide arms through the shoulder straps like a backpack and adjust the shoulder straps to a comfortable position.

**Step 4.** Wrap the left side panel to the front to position the front panel. Then, wrap the right side panel on top of the left panel to close. Attach both sides in front. Adjust as needed.

**Step 5.** Grasp the pulley handles and pull out and away from the body to reach the desired compression and support. Fasten handles to the waist panels to secure.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

