

# Air Walker CAM Fracture Boot - Short

SKU: BTS-AD320

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Open the liner and insert your foot with the heel firmly in the back of the boot.

**Step 2.** Wrap the liner around the foot and fasten the Velcro.

**Step 3.** Fasten foot straps.

**Step 4.** Feed top strap through D Ring and fasten.

**Step 5.** To inflate the liner, first, ensure the deflation valve is closed. Press the blue air pump repeatedly until comfortable.

**Step 6.** To deflate the liner, turn the deflation valve and squeeze the liner to remove any air.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Spray with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

