

# Cam Walker Fracture PDAC Boot Short

SKU: BTS119

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code




## Instructions For Use

**Step 1.** Place foot and leg inside foam liner and fasten it snugly.

**Step 2.** Add additional pads to supplement inner space as needed.

**Step 3.** Place liner and foot into walker boot.

**Step 4.** Fasten ankle, foot and calf strap in order. Tighten straps til snug and comfortable.

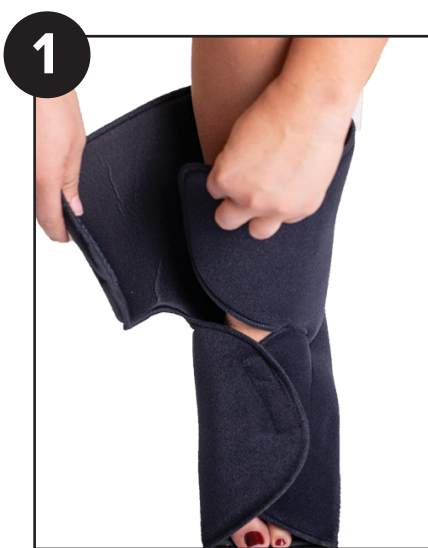

Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach. 