

Wide Air Cam Fracture Boot Tall Extended Width 2E

SKU: BTS219

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Place foot and leg inside foam liner and fasten it snugly.

Step 2. Place liner and foot into walker boot.

Step 3. Add additional pads to supplement inner space as needed.

Step 4. Fasten ankle, foot and calf strap in order. Tighten straps until snug and comfortable.

Step 5. Adjust aircell pressure by squeezing the bulb to inflate.

Step 6. To deflate, press the valve below the inflation bulb.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

