

# Wide Cam Walker Fracture Boot Tall - Extended Width 2E

SKU: BTS221

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Place foot and leg into the foam liner, securing it snugly.

**Step 2.** Spread uprights and insert the liner-covered foot into the boot ensuring the heel is firmly against the back.

**Step 3.** Add supplemental pads to fill any gaps for a secure and comfortable fit.

**Step 4.** Fasten the ankle, foot and calf strap in sequence, adjusting each until comfortably snug.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

