

Pediatric Walker Fracture Boot

SKU: BTS249

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code




Instructions For Use

Step 1. Place foot and leg inside foam liner and fasten it snugly.

Step 2. Add additional pads to supplement inner space as needed.

Step 3. Place liner and foot into walker boot.

Step 4. Fasten ankle, foot and calf strap in order. Tighten straps til snug and comfortable.

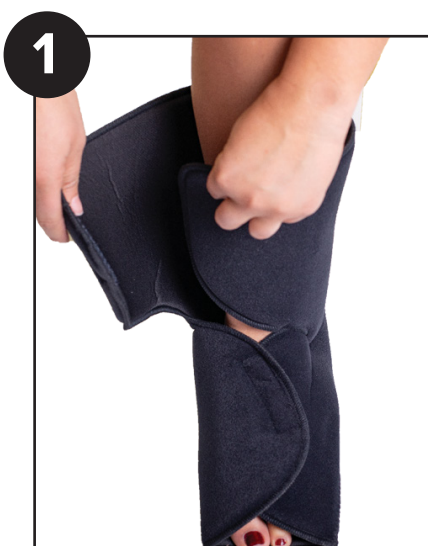

Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach. 