

Lightweight Medical Full Shell Walking Boot with Air Pump

SKU: BTS295

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Apply in a seated position. Unfasten Velcro straps and remove liner. Place foot in liner and fasten Velcro.

Step 2. Spread uprights and place foot and liner into boot. Foot should be flat with heel in the back of the boot. Uprights should be parallel to the leg.

Step 3. Close straps starting at the toes and work up the leg.

Step 4. To inflate the liner, use the bulb on the front of strap. Inflate to comfort level. Do not overinflate. Check circulation to ensure it is not too tight.

Step 5. To release air, turn white knob and squeeze liner.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

