

# Short Air Walker Boot with Pink Liner

SKU: BTS296

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Place foot and leg inside foam liner and fasten it snugly.

**Step 2.** Place liner and foot into walker boot. Fasten ankle, foot and calf strap in order. Tighten straps until snug and comfortable.

**Step 3.** Adjust aircell pressure by squeezing the bulb to inflate.

**Step 4.** To deflate, press the valve below the inflation bulb.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

