

# Tall Air CAM Walker Boot

SKU: BTS315

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** In a seated position, place foot and leg inside foam liner and fasten it snugly.

**Step 2.** Add additional pads to supplement inner space as needed.

**Step 3.** Place liner and foot into walker boot.

**Step 4.** Fasten ankle, foot and calf strap in order. Tighten straps until snug and comfortable.

**Step 5.** To inflate, press air valve until comfort level is reached.

**Step 6.** To deflate, turn the valve stem until liner is deflated.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

