

# Plantar Fasciitis Posterior Night Splint

SKU: FTB133

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code




## Instructions For Use

**Step 1.** Unfasten all straps and place foot in splint with knee flexed to achieve desired degree of dorsiflexion.

**Step 2.** Attach ankle strap and adjust. Attach foot strap and calf strap. Adjust as needed.

**Step 3.** Adjust dorsiflexion assist straps. Do not walk in splint.

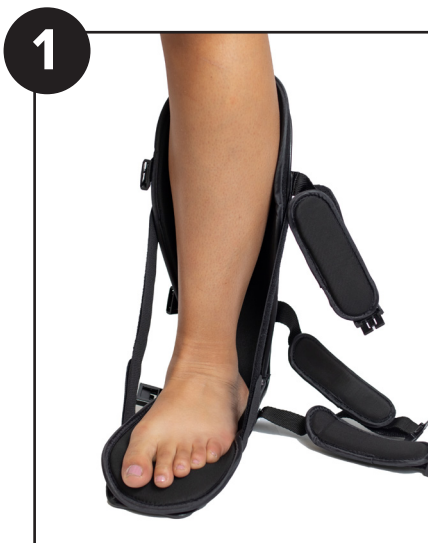

Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach. 