

# Hybrid Night Splint Orthotic Brace

SKU: FTB162

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Unfasten straps on one side. Place foot in splint so that toes are near outer edge of the foot pad.

**Step 2.** Fasten straps after ensuring a comfortable fit.

**Step 3.** Use front adjustable strap to adjust to your desired angle.  
Do not over tighten.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

