

# Leaf Spring Semi-Rigid AFO

SKU: FTB212

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

### Step 1. Prepare the Brace


Unstrap the velcro and place your heel securely in the back of the brace.

### Step 2. Adjust the AFO

Mark the length of your foot and trim the AFO with scissors to desired length.

### Step 3. Secure Foot

Insert AFO into shoe, then place foot into shoe. Secure Velcro strap around the calf. Trim Velcro strap if needed.

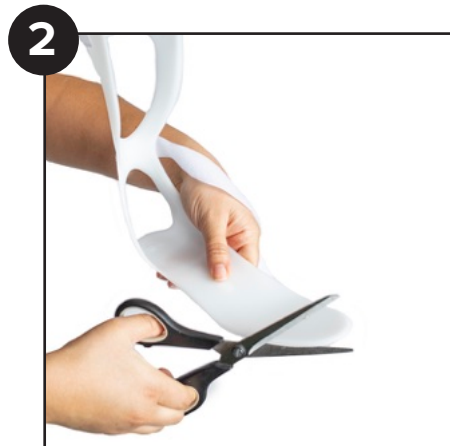

Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach. 