

Adjustable Night Splint PDAC

SKU: FTB255

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code




Instructions For Use

Step 1. Unfasten all straps and place foot in splint with knee flexed to achieve desired degree of dorsiflexion.

Step 2. Attach ankle strap and adjust. Attach foot strap and calf strap. Adjust as needed.

Step 3. Adjust dorsiflexion assist straps. Do not walk in splint.

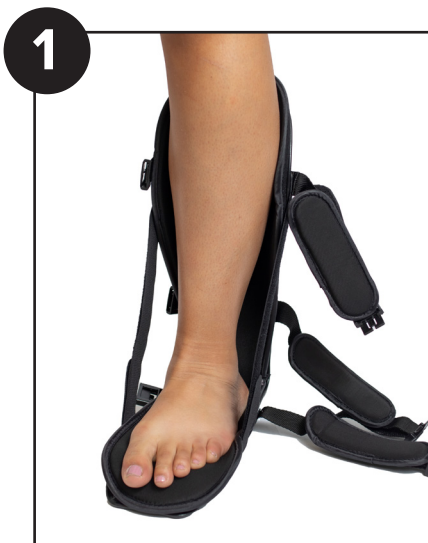

Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach. 