

Swedish Leaf Spring AFO Orthosis Drop Foot Brace

SKU: FTB302

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Prepare the Brace


Unstrap the velcro and place your heel securely in the back of the brace.

Step 2. Adjust the AFO

Mark the length of your foot and trim the AFO with scissors to desired length.

Step 3. Secure Foot

Insert AFO into shoe, then place foot into shoe. Secure Velcro strap around the calf. Trim Velcro strap if needed.

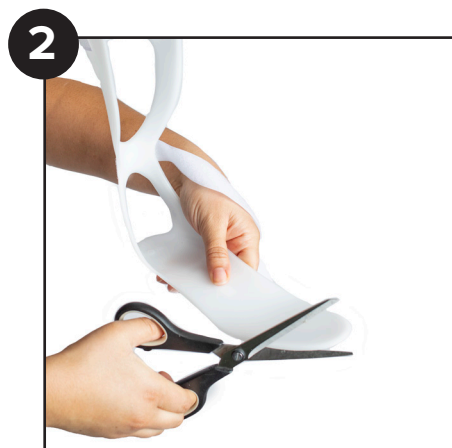

Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach. 