

# ACL ROM Ligament Knee Brace

SKU: KNB-CL-CK-207

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Slip foot through the brace so that the frame is behind the calf and in front of the thigh.

**Step 2.** Center the ROM Hinges on the mid-line of the knee.

**Step 3.** Beginning with the bottom calf strap, secure each strap to fit snugly, ensuring the second from bottom calf strap is sitting above the calf muscle to prevent sliding.

**Step 4.** To adjust the ROM settings, pull the extension or flexion tab out and move the brace frame into the desired position. Push the tab back in to set.

**Step 5.** If you need to lock the brace so that there is no movement. Locate the tab just above the ROM hinges and pull it down into lock position. Ensure both sides are locked. ROM hinge settings should be provided by your physicians.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse thoroughly and Air Dry.
- Do NOT dry clean, iron, or bleach.

