

# Dual Hinged ACL Ligament Knee Brace

SKU: KNB-CL-ER-208

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Unfasten all the straps on the brace and place the leg into the brace.

**Step 2.** Position the brace so that the hinges are centered on the sides of the knee.

**Step 3.** Adjust straps to fit and fasten all straps into place, ensuring the calf strap is above the calf muscle to prevent sliding.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Spray with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.

