

Knee Brace Undersleeve

SKU: KNB198

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Open the undersleeve, locate the larger opening ensuring it is on top, and slide your foot through the sleeve.

Step 2. Pull the undersleeve up the thigh.

Step 3. The sleeve can be folded over to shorten the length if needed. The brace can then be applied over the sleeve.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Machine wash in mild temperature.
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.   

