

Tri Panel Knee Immobilizer

SKU: KNB235

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Unfasten all straps and open the immobilizer. Position the brace at the bend of the knee.

Step 2. With knee cap centered, secure the first Velcro strap just below the knee.

Step 3. Secure the remainder of the straps in an alternating fashion until all are fastened.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

