

Anti-Slip Compression Knee Sleeve

SKU: KNW214

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Remove the suspension wrap from the package and unfasten the Velcro tab to open. Wrap around the portion of the calf where the bottom of the brace sits. Secure with Velcro tab.

Step 2. Place the knee brace on the leg, ensuring the bottom of the brace is sitting on the suspension wrap.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Spray with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

