

Heated Knee Massage Therapy Unit

SKU: KTU256

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Attach the straps by sliding the plastic clips onto the tabs on the side of the therapy device.

Step 2. Insert the plug into the device, located near the control panel.

Step 3. Place the device on the knee with the control panel facing the body.

Step 4. Wrap the straps around the leg and secure the clips to the tabs on the other side of the device.

Step 5. Adjust your treatment settings to the desired specifications.

Step 6. Cycle through settings by pressing the function buttons until the desired settings are achieved.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Spray with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

