

# Deluxe Post-Op Shoe L3260

SKU: POS223

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** While seated, unfasten all straps. Place heel at the back of the shoe and secure the ankle strap.

**Step 2.** Fasten toe straps, ensuring toes are not over the end of the shoe. Adjust as needed to accommodate bandages or swelling. Do not over-tighten.

**Step 3.** Trim excess ankle strap as needed.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse thoroughly and Air Dry.
- Do NOT dry clean, iron, or bleach.  

