

Frozen Heat

SKU: UCTY159

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Prepare the Machine

Begin by pressing the red button to open the machine. For cold therapy, fill with ice up to the marked line and add water to the designated water line. For heat therapy, add hot water to the fill line. Securely close the lid by pushing it down until you hear a distinct click.

Step 2. Connect Thermal Tubing

To attach the thermal tubing, press the metal clamps on the ports located on the back of the unit. Insert the female end of the tubing into the ports until you hear a secure click.

Step 3. Attach Universal or Accessory Pad

Connect the universal or accessory pad by pressing the metal clamps down on the other end of the thermal tubing. Insert the universal pad fittings until you hear a secure click.

Step 4. Power On

Locate the power port on the back, near the tubing. Plug in the AC adapter and turn on the device using the power switch.

Step 5. Apply the Pad

Place the pad with the plastic side down and the gray side facing away. Apply it to the affected body part and use the elastic wraps to secure the pad in place.

Step 6. Set Treatment Parameters

Press and hold the Mode (M) button to choose your treatment settings. Use the arrow buttons to adjust the flow rate and treatment time. Press Mode again to finalize your settings. Commence the therapy by pressing the start/pause button, and you can pause it at any time by pressing the same button.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

