

Premium ROM Wrist Brace Universal

SKU: WRB217

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Unfasten all the straps and open the inner adjustable material. Slide hand and wrist into the brace with the thumb through the thumb hole.

Step 2. Tighten and secure the two fastening straps closest to your elbow first. Take hold of the last remaining fastening strap and pull it over your wrist on a slight downward angle, then cross back over the lower portion of your thumb and secure the fastening strap into place.

Step 3. Loosen the Range of Motion knob located at your wrist and position your hand as instructed by your physician. Secure into place rotating the wheel clockwise until it is tight.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.   

