

# Carpal Tunnel LYCRA Wrist Brace Support

SKU: WRB228

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Unfasten the Velcro tabs and open the brace.

**Step 2.** Insert hand into brace ensuring the beaded support pillow is under the wrist.

**Step 3.** Fasten all of the Velcro tabs for a snug fit.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Spray with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

