

Pediatric Wrist Brace

SKU: WRB299

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



Instructions For Use

Step 1. Open brace and position hand in wrist brace.

Step 2. The palmer stay should line up below the middle crease of the hand.

Step 3. Secure the thumb strap to the brace, aligning it between the thumb and index finger.

Step 4. Wrap the lacing around the wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.



Still need help?
Perfect Fit Guarantee!

Visit our website or scan the QR code to find out more.

Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse and Air Dry.
- Do NOT dry clean, iron, or bleach.  

