

# CMC Osteoarthritis Thumb Ring Brace

SKU: WRW105

Prefer watching videos?

We've got you covered!

Visit our YouTube @BraceDirect or scan the QR code



## Instructions For Use

**Step 1.** Unfasten the Velcro strap and slide the brace over your thumb. The aluminum stays should be at the base of your thumb.

**Step 2.** Place the Velcro through the guide loop and fasten for a secure fit.

**Step 3.** Form the aluminum stays to your thumb. These can be reformed as needed.



Still need help?  
**Perfect Fit Guarantee!**

Visit our website or scan the QR code to find out more.

### Care Instructions

- Hand Wash with a 50% Alcohol/Water Mixture
- Rinse thoroughly and Air Dry.
- Do NOT dry clean, iron, or bleach.  

