

Being smart is just the start.
Choose Smartee

Smartee[®]

Clear Aligner



Exclusive distributor of Smartee[®] in the Philippines

tesco

DENTAL
SUPPLIES
LIMITED

PREDICTABLE EFFECT FOR A BEAUTIFUL SMILE

A close-up portrait of a young woman with dark hair pulled back, smiling warmly. She is wearing blue medical scrubs over a grey t-shirt. In the bottom left corner, she is holding a white tablet computer. The background is a soft, out-of-focus light blue and white.

The treatment effects can be foreseen in advance through 3D computer imaging technology, which can intuitively and accurately present the correction process by animation simulation.

THE BENEFITS OF Smartee®

Clear Aligner



Enjoy a variety of foods
and get balanced nutrition

Take it off at any time,
no pressure on any occasion



Can be removed at
any time for cleaning

Normal use of toothbrush
and interdental brush
to maintain oral health





APPLIANCE WEARING TIME

Wear the aligner **22** hours a day

Replace the aligner with a new one every **2** weeks

(Please follow your dentist's advice for specific wearing time)



Rinse the aligner
with water
before wearing



The aligner can be
removed before eating,
brushing and flossing



Brush your teeth
with a soft
toothbrush



The aligner can
be cleaned with an
ultrasonic cleaner

FOR ILL-FITTING OR LOST APPLIANCE

Try the next aligner

DOESN'T FIT

Please wear the former aligner
and contact your dentist.

PROPER FIT

Please continue to wear it.



QUESTIONS YOU MAY BE CONCERNED ABOUT

Q How quick is the treatment with Smartee Clear Aligner?

A The actual orthodontic treatment time varies from person to person. Generally, adults need 12-28 months to complete the treatment.

Q How does Smartee Clear Aligner move teeth?

A Smartee Clear Aligner will apply an orthodontic force to the teeth so that the teeth can move within a reasonable range according to the doctor's orthodontic treatment plan.

Q Will my teeth hurt during the invisible orthodontic treatment?

A The teeth will have a slight soreness in the first two to three days, and then have no obvious sensation in the rest of the time.

Q What should I remember for best results with Smartee Clear Aligner?

A You should remove the aligner before eating or brushing teeth, but you should wear it no less than 22 hours a day. The more cooperative you are, the better your treatment effect is.

Q How to clean the aligner routinely?

A It is recommended to rinse it with warm or cold water. Do not clean it with hot water or corrosive detergent.

Q What should I do if the aligner is lost?

A If you lose the aligner accidentally, you can wear the aligner at the previous stage and contact your dentist.

Q Are adolescents suitable for invisible orthodontic treatment?

A As long as permanent teeth are erupted, adolescents are suitable for treatment with Smartee Clear Aligner. In general, you should return for follow-up visit every 4-6 weeks. Wearing the clear aligner is more conducive to keep oral health and to reduce caries. The aligner is transparent and does not affect social interaction.



Being smart is just the start. Choose Smartee

Smartee®

Clear Aligner



CUSTOMIZED TREATMENT WITH CLINICAL MONITORING PROVIDED BY DENTISTS

Customized special solutions based on individual orthodontic data.
Customized special aligners for you by 3D printing technology.



For more details, you may contact:

Chris Dalida Guzman, DMD - Professional Affairs Specialist: 09158022618

SMR012422_500PH