

Congratulations on your new DeBug Elevating Legrest Beach Wheelchair



DeBug Mobility Products



78 E Nine Mile Rd
Pensacola, FL 32534

Corporate Address
Deming Designs, Inc.
1090 Cobblestone, Dr.
Pensacola, FL 32514

2

TABLE OF CONTENTS

Usage and Safety Information	3
General Maintenance Instructions	
Assembly / Disassembly of DeBug Beach WheelChair	4 - 5
General Specifications and Warranty	6

Thanks for your purchase of the
DeBug Beach Wheelchair

We build the DeBug Beach Wheelchair with the highest quality materials, keeping in mind the comfort and positioning of the rider. If any repairs should be required, please do not attempt them without proper technical training. Please contact your nearest distributor for any repairs, or adjustments needed. All information in this manual is subject to change without prior notice. Please read the following few pages for Assembly and Maintenance Instructions. Thank-you The De-Bug Mobility Products Team. We love to hear and see how much you are enjoying your DeBug Beach Wheelchair. Feel free to email us any pictures or videos, or post them to our website.

www.beachwheelchair.com

Corporate Address

Deming Designs, Inc.
1090 Cobblestone, Dr.
Pensacola, FL 32514

www.beachwheelchair.com

850-478-5765

3

USAGE & SAFETY

DeBug Beach Wheelchair should not be used during inclement weather, dangerous surf, rough ocean currents, rip tides and any other poor conditions. !

We recommended the chair be used only when a lifeguard is present. Safety comes first! Users must always be accompanied by a responsible, able-bodied adult.

Using your DeBug Beach Wheelchair

Always maintain control of your DeBug while in use. When the DeBug is not in use, the safety brakes must be engaged to prevent the chair from moving unexpectedly. The DeBug was built to not tip over, even when used over uneven surfaces. However, to ensure safety and prevent injury, the user must always wear the seat-belt while using the DeBug. This is imperative to the user's safety.

Do not use the components of the DeBug Beach Wheelchair for any purpose other than what is intended. Do not have more than one person using the DeBug. Review Safety on Warnings Before Using your DeBug Beach Wheelchair

General Maintenance

It's so easy to maintain your DeBug Beach Wheelchair! Just follow these simple instructions, and you will be able to enjoy your chair year after year.

Rinse the chair off with clean water to remove any sand or debris after each use. The sling material is removable and machine washable.

Check tire pressure before each use

If at any time you need to lubricate any part of the chair, DO NOT USE WD-40, as it will attract dirt and sand, causing possible damage to the wheel bearing/ bushings. We recommend P.T.F.E Dry Lube, all wearing components. This dry film is a superior lubricant and repels debris and water, marine and performance parts stores likely stock this locally.

Have any questions, or need additional support? Please don't hesitate to contact us!

Corporate Address

Deming Designs, Inc.
1090 Cobblestone. Dr.
Pensacola, FL 32514

4

Assembly / Disassembly of DeBug Beach WheelChair



1/4-20x1-1/4" SS Hex Bolt
2 each for Backrest

Removable Armrest

1/4-20x2-1/4" SS Hex Bolt
4 each for Anti-Tip feature

1/4-20x2-1/2" SS Hex Bolt
2 each for Main Axle



1/4"- 20tp X 1-1/4" SS
Backrest Bolts
2 each



5/16"-18tp X 1-1/4" SS
Rear Suspension Bolt
1 each



Legrest Adjustment Knobs
2 each



1/4 - 20 X 2,25" SS Hex Bolts
2 each



18 SS Main Axle Pin
2 each



1/4" - 29tp x 2-1/4"
SS Anti-Tip Bolt
4 each



1" SS Main Axle Flat Washer
2 each

Corporate Address
Deming Designs, Inc.
1090 Cobblestone, Dr.
Pensacola, FL 32514

5

Assembly / Disassembly of DeBug Beach WheelChair

ASSEMBLING INSTRUCTIONS



Legrest in the lowered position



Legrest in Raised Position

1. Slide on the leg support sling, and install the two 5/16"-18 adjustment knobs
2. Slide on the foot plate, and tighten the leg rest adjustment knobs
3. Install the mid axle, tighten the two attaching 1/4"-20 bolts
4. To install the backrest, slide the backrest down on the chair frame and attach using the two 1/4"-20 bolts
5. The rear suspension may be attached by inserting the rear suspension tension bolt. This bolt should be tight during use

DISASSEMBLING INSTRUCTIONS

1. Loosen the leg length adjustment knobs and slide out the footplate
2. Slide off the leg support sling by removing the two 5/16"-18 adjustment knobs
3. To remove the mid axle, loosen the two attaching 1/4"-20 bolts
4. To remove the backrest, remove the two attaching 1/4"-20 bolts and slide up on the backrest
5. The rear suspension may be detached by removing the rear suspension tension bolt. This bolt should be tight during use

Corporate Address

Deming Designs, Inc.
1090 Cobblestone, Dr.
Pensacola, FL 32514

6

GENERAL SPECIFICATIONS AND WARRANTY INFORMATION

30 DAY

Money Back Guarantee
Less Shipping and possible Restocking Fees

2-Year

Non-absorbent, durable U.V. resistant Phiffertex Plus brand nylon seating-material. Removable and machine washable seating.

10-Year

Components & Frame #316L series stainless steel The higher the number the higher quality of stainless

The DeBug Beach Wheelchair frame is constructed using #316L Stainless Steel tubing, which is a higher grade than standard “#304 marine grade” stainless. Weight capacity 350 pounds.

The rear Caster wheels rotate 360 degrees. The wheel hubs are made from poly-nylon material to eliminate saltwater corrosion.

The DeBug has a footrest for proper foot support.

Wheel Lock –Adjustable tension wheel brake can be actuated independently on each side,



DEBUG
MOBILITY PRODUCTS

78 E Nine Mile Rd
Pensacola, FL 32534

Corporate Address

Deming Designs, Inc.
1090 Cobblestone, Dr.
Pensacola, FL 32514

www.beachwheelchair.com

850-478-5765