

# Flexineb C2 Firmware Update Instructions

## Step 1:

Connect the supplied USB cable to the C2 controller and a Windows PC or Laptop as shown.




## Step 2:

Go to [www.flexineb.com](http://www.flexineb.com) website, click on **Support** tab - **Firmware Update**

Or click this link <https://flexineb.com/pages/firmware-update>

## Step 3:


Click on **Download (Windows)**.

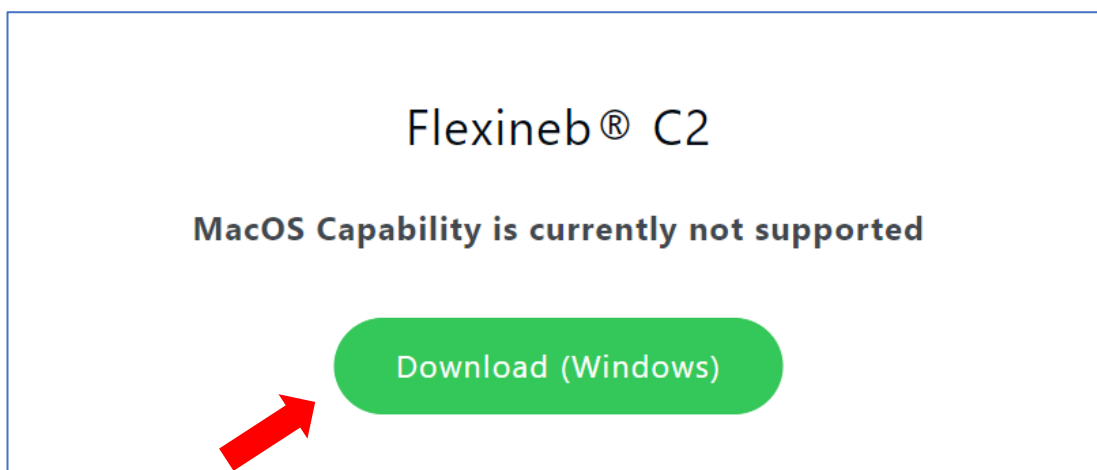

# Flexineb C2 Firmware Update Instructions


## Step 4:

An executable file will download to the bottom left corner of your PC.

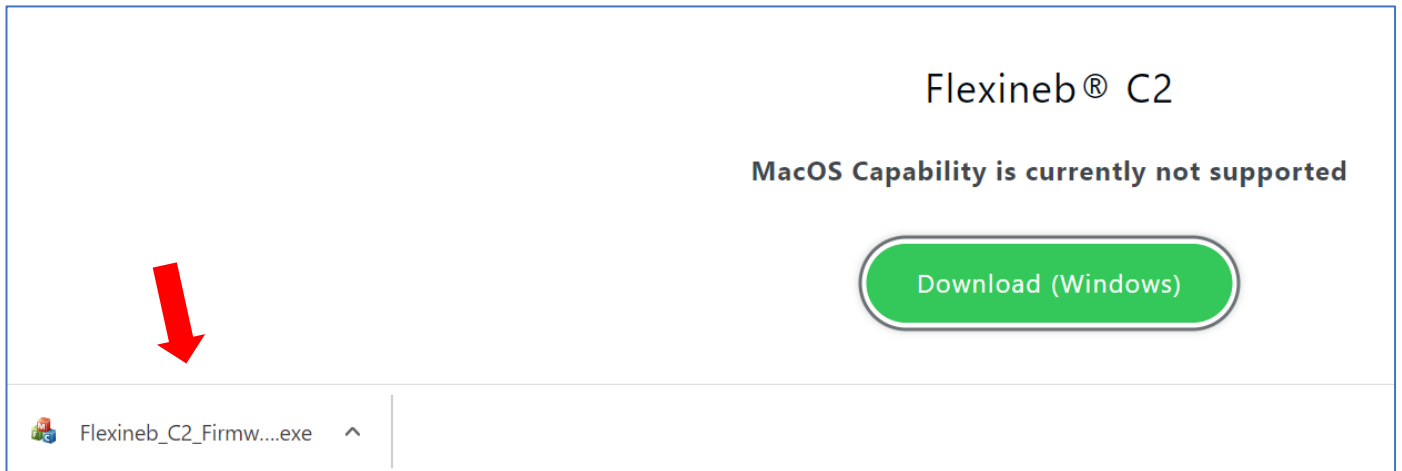


## Step 5:

Click on the **^** arrow and then **Open**.

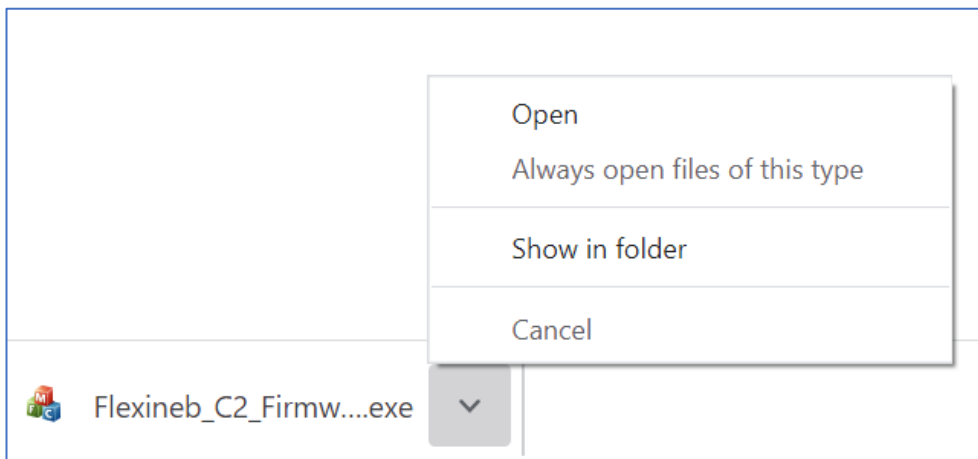


## Step 6:

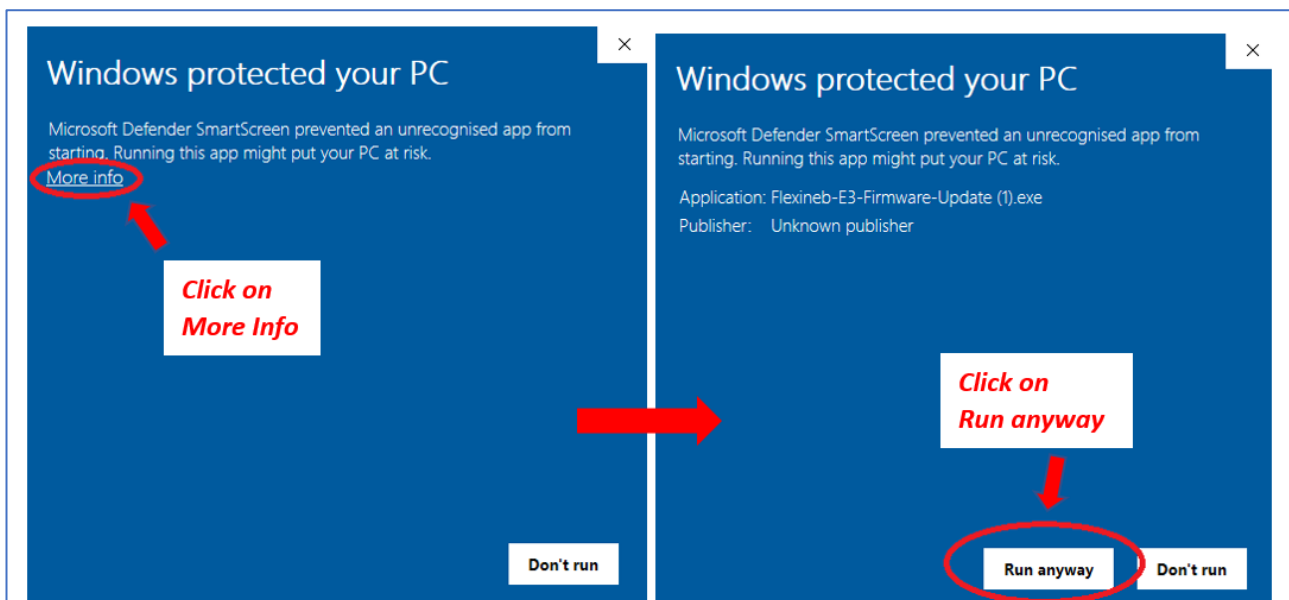

## Flexineb C2 Firmware Update Instructions


### Step 7:

The program will search for your C2 unit before proceeding to the **Update** function.  
Click on **Update**.

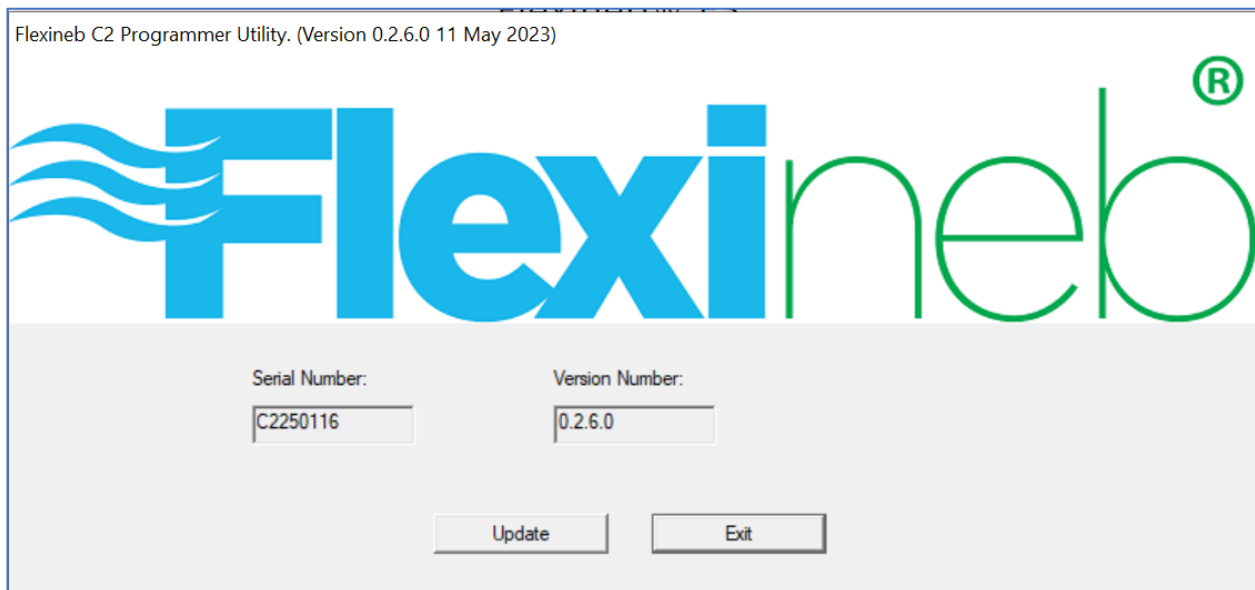

### Step 8:

**Device updated successfully** message appears.  
Click **OK** and then **Exit** to complete the update.

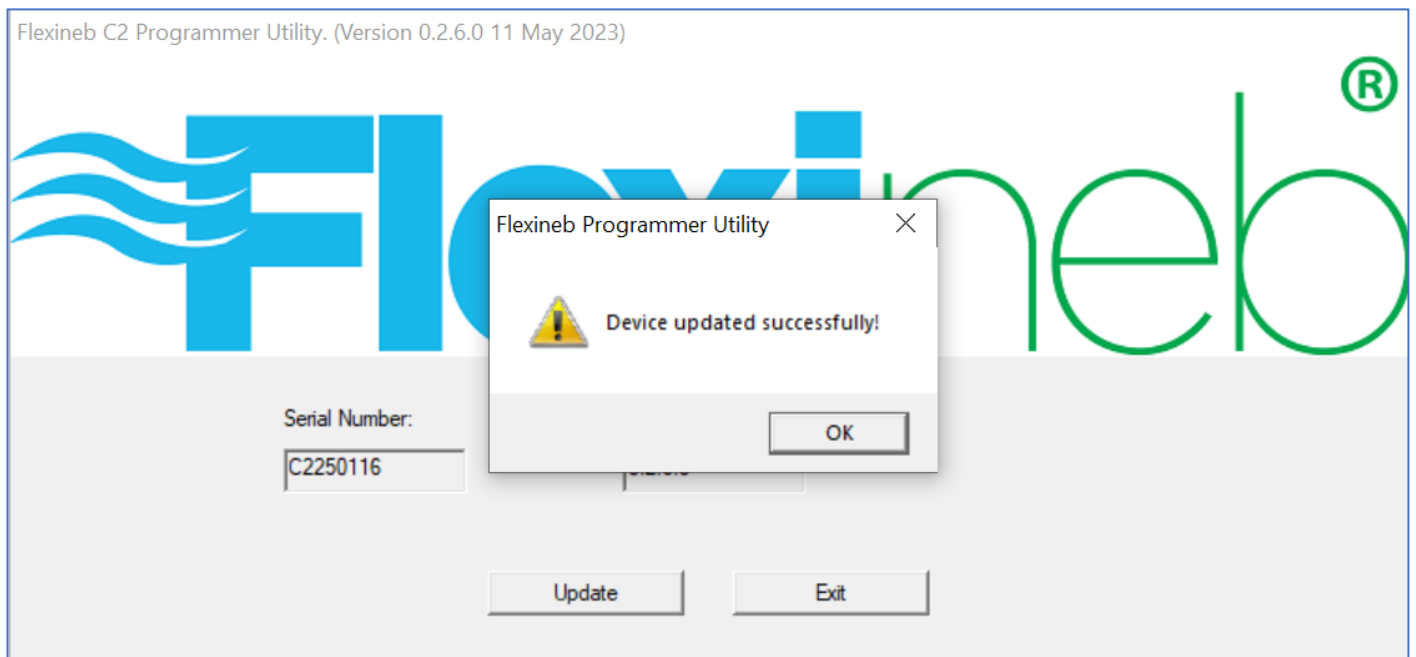

Unplug the USB cable to complete the update process.

**END**