

**GATE**

REHAB DEVELOPMENT AB™



# Bure Rise & Go™

StandTall walker with power rise

**New unique concept!**

# Bure Rise & Go™

## StandTall walker with power rise

The Bure Rise & Go is a cost-effective walker enhanced with a patented power rise function. Thanks to the electric power rise function, Bure Rise & Go acts as a combined stand-up, mobility and walking aid all in a single product.

**New** – Rise & Go is now also available with electric walking frame widening. Flexible widening combined with the lower height above the walking frame enhances versatility regardless of the patient's situation in hospital or at home.



### Step 1

The harness is fitted over the patient's lower back and bottom, the straps adjusted and the leg supports placed in the correct position.

### Step 2

The patient holds on to the handles and the electrically driven power rise function begins. The patient follows a natural pattern of movement with the aid of the unique harness and the vertical lift action.

### Step 3

On completion of the power rise, the leg supports are swung away and the patient is free to walk using the StandTall walker. Alternatively, patients place their feet on footplate and are ready to be moved once power rise is complete.

## Unique power rise assistance

Bure Rise & Go saves time, space and money. It acts as a stand-up, mobility and walking aid in one and the same product, which makes the concept extremely cost effective.

The Bure Rise & Go enables better work ergonomics for carers, trainers and anyone else working with patients. Thanks to the smart power rise assistance, standing up becomes a safe, confident and natural action that does not wear out the backs of health care personnel.

The product consists of a walker and its associated harness and straps. Harnesses are available in different sizes (S, M, and L) and they are split at the rear to make e.g. WC visits easier.

When the patient stands up the electric power rise function is employed and the attachment reduces the distance between the patient and the walker until the walker is in the standing position. The Bure Rise & Go contributes to reduced workloads and is an investment in personnel health and the patient's rehabilitation.

**New** – Rise & Go is now also available with electric walking frame widening. Flexible widening combined with the lower height above the walking frame enhances versatility regardless of the patient's situation in hospital or at home.

**Stable**

The walker has anatomical cushions that provide relief for shoulders, arms and the neck and shift the focus to larger muscle groups such as leg and chest muscles instead. This improves stability and makes the walker easier to steer.

**Easily reachable hand controls**

The hand control is located such that both of the carer's or user's hands can easily reach them.

**Ergonomic handles**

Ergonomically designed handles that are steplessly adjustable in every direction for optimum individual adaptation. This is extremely important for stroke patients who can thus reach their "usual" hand position.

**Folding armrests**

Bure Rise & Go's anatomically designed armrests can be folded out to provide the user with an easy means of reaching the handles during the standing manoeuvre.

**Flexible stepless adjustment**

Electrically driven height adjustment makes it easy to pre-set the walker to the desired height. Once the patient is standing in position the height can be steplessly adjusted for comfort and the proper support for an optimal walking configuration.

**Sustainable**

The walker is designed to last a long while. For example it has extra reinforcement bands around the arm rest cushions to prevent them from splitting from impacts to the side.

**Power Rise**

Bure Rise & Go is a unique aid for assisting patients or users during the standing maneuver. Personnel burdens, and thus repetitive strain injuries, are minimized.

**Unique power rise harness**

The power rise harness is designed to reach as far down across the bottom as possible. This provides optimum support which results in a natural standing action.

**Waterproofing**

All Bure Rise & Go electrical components have IP 65 international protection ratings and can therefore be used in wet spaces.

**Leg support for the standing manoeuvre**

Bure Rise & Go is fitted with leg supports that make for a better standing maneuver; they are height adjustable and can be moved aside simply to create greater space.

**Low height**

The low height above the floor minimizes problems such as getting close to the user e.g. when he or she is sitting on a bed.

**Lockable castors**

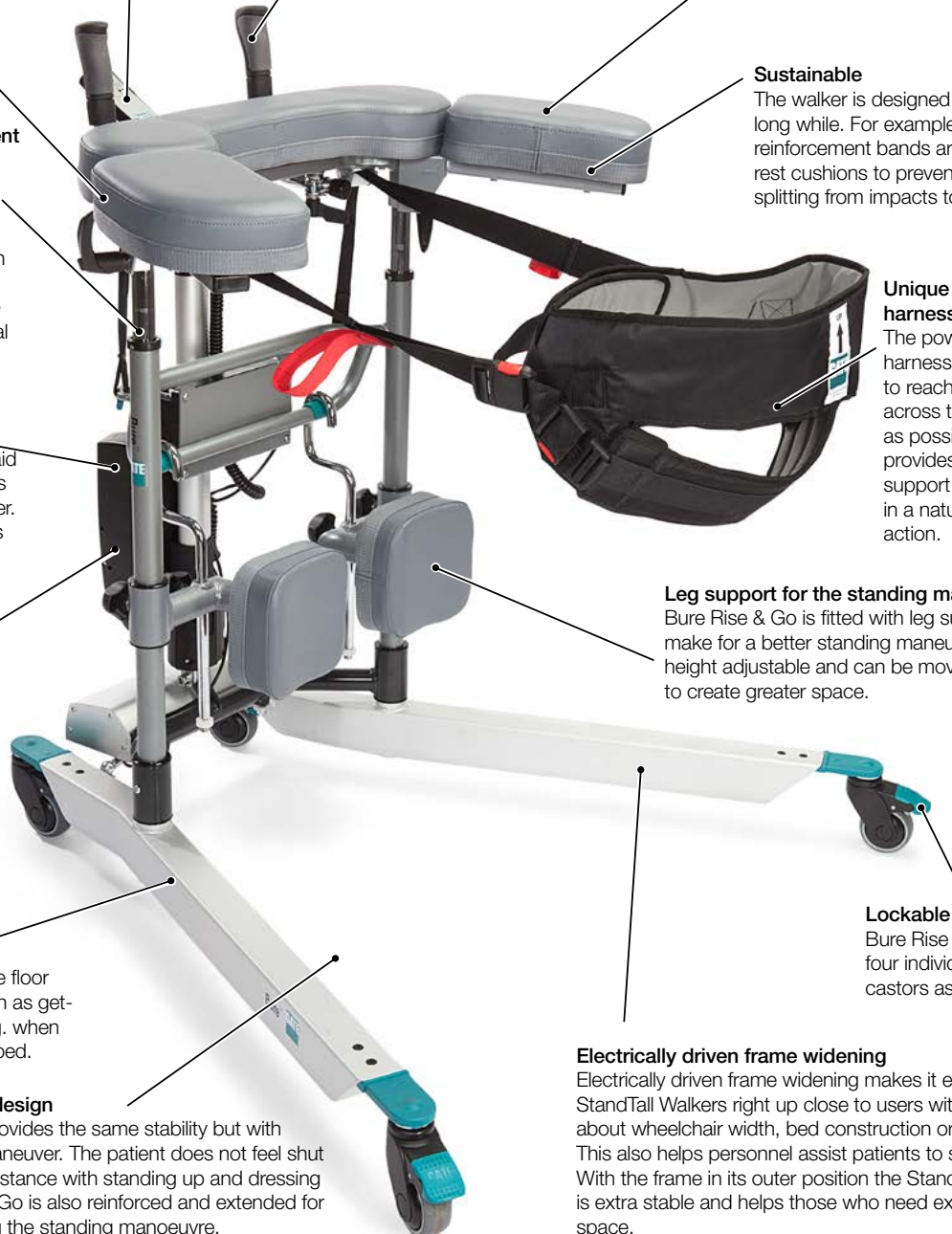
Bure Rise & Go has four individually lockable castors as standard.

**Open, reinforced design**

The open design provides the same stability but with greater room for maneuver. The patient does not feel shut in and it makes assistance with standing up and dressing easier. Bure Rise & Go is also reinforced and extended for extra stability during the standing manoeuvre.

**Electrically driven frame widening**

Electrically driven frame widening makes it easy to move StandTall Walkers right up close to users without worrying about wheelchair width, bed construction or seat width. This also helps personnel assist patients to stand up. With the frame in its outer position the StandTall Walker is extra stable and helps those who need extra walking space.

**Technical specifications**

CE Tested and approved according to SS-EN 12182:1999

Part No.		Width, external	Max user weight	Length	Height	Armrests/inside dimension
56-309/75	Bure Rise & Go DB with electric frame widening.	66-106 cm	150 kg	101 cm	89-129 cm	34-67 cm
56-316	Bure Rise & Go StandTall walker	77 cm	150 kg	92 cm	96-136 cm	34-67 cm
56-333-DB	Footplate short		150 kg			
56-383-DB	Footplate long		150 kg			
56-384-S/M/L/XL	Harness (available in size S, M, L, XL*)	-	150 kg	-	-	-
56-388-S/M/L/XL	Gait training harness (available in size S, M, L, XL*)	-	150 kg	-	-	-

\* S=95-105cm, M=105-120cm, L=110-140cm, XL=130-160cm

# Bure Rise & Go™

StandTall walker with power rise

- **BURE RISE & GO** is a complete solution that saves time, space and money which means great cost efficiency.
- **SAFE, CONFIDENT** standing and mobility mean a better working environment for health care personnel.
- **THE UNIQUE** power rise assistance provides a natural standing action that allows the patient to participate.
- **BURE RISE & GO** is a patented, protected proprietary GATE Rehab Development AB design.
- **THE WALKER CAN** be used with a footplate that makes patient transits easy.
- **THE HARNESS IS WASHABLE** at 60°, and thanks to its split design it is simple and flexible to wear even during visits to e.g. the WC.
- **BURE RISE & GO** is equipped with a smooth, steplessly variable electric height adjustment.
- **NEW** - Rise & Go is now also available with electric walking frame widening for added flexibility regardless of the user's surroundings.

The footplate is ideal for use in patient transits!



Watch the Bure Rise & Go video at [www.gaterd.se](http://www.gaterd.se)

## Gait Training Harness

(accessories)

Gait training harnesses are accessories used for increased safety when training using the Bure Rise & Go. The product's two groin straps and attachment hooks minimize the risk if the user becomes dizzy or similar.



# GATE

**WE DO BETTER.**

Would you like to know more?

Contact us at: tel +46 371-318 00  
Gate Rehab Development AB  
Industrigatan 2 | SE-330 21 Reftele  
info@gaterd.se | www.gaterd.se



GATE COLLECTION