


# Syringe Adapter Kit Installation

(Model #01-20)

- Step 1-** Using a hexagonal allen wrench, screw threaded adapter nut (A) into syringe head. Be certain that the o-ring (C) is securely placed in the end of the adapter nut before tightening.
- Step 2-** Place o-rings (D) within the chrome tip holding nut (B).
- Step 3-** Using the allen wrench, screw chrome nut assembly onto adapter nut (A).
- Step 4-** Insert the syringe tip (E) by pushing into the chrome nut (B).

**NOTE:** For best results use #01-02 autoclavable syringe tips.

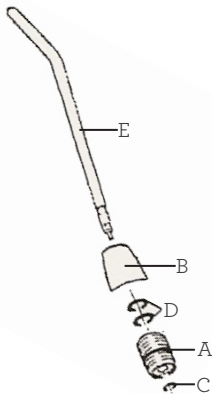

**AMERICAN**dental

accessories

7310 Oxford Street • St. Louis Park, MN • 55426

Phone (800) 331-7993 • Fax (888) 729-1016

[www.amerdental.com](http://www.amerdental.com)