

# Air/Water Syringe Installation

(Model #'s 01-81, 01-89, 01-82, 01-83, 01-01Q, & 01-90D)

**Step 1-** Shut unit off and relieve pressure in existing syringe by depressing both buttons until flow stops.

**Step 2-** Disconnect existing syringe by unscrewing handle & cutting the tubing directly below the syringe head. Take care to note which tubing is water and which is air.

**Step 3-** Insert tubing through handle, and slip sleeve clamps over each line approx. 3/8" from end.

**Step 4-** Slip tubing over barbs and lock tubing by sliding sleeve clamps over tubing portion on the barb with a pliers or sleeve clamp tool. Turn unit on and test for leaks and proper function.

**Note:** Take care not to break barbs when installing tubing and clamps. Inserting tubing ends in hot water will soften the tubing making it easier to slide it over the barbs.



**Warning-** Always insure syringe tips are securely attached before use.



7310 Oxford Street • St. Louis Park, MN • 55426  
Phone (800) 331-7993 • Fax (888) 729-1016  
[www.amerdental.com](http://www.amerdental.com)