


Lever Style HVE Valve Installation

(Model #'s 03-06, 03-07, 03-12 & 03-14- Non-Autoclavable)

- Step 1-** Turn off vacuum.
- Step 2-** Cut existing valve from tubing.
- Step 3-** Attach the valve to the tubing.
Inserting tubing end in hot water
and/or the use of soap will aid in
this process.

NOTE: After each disinfection- disassemble, clean
& flush thoroughly with water & lubricate
all moving parts with a silicone based
lubricant.

DO NOT USE BLEACH ON THIS PRODUCT.
DO NOT USE HEAT STERILIZATION.

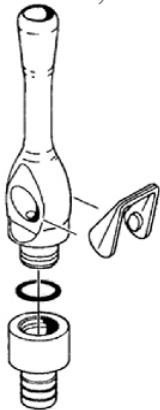

AMERICANdental
accessories

7310 Oxford Street • St. Louis Park, MN • 55426

Phone (800) 331-7993 • Fax (888) 729-1016

www.amerdental.com